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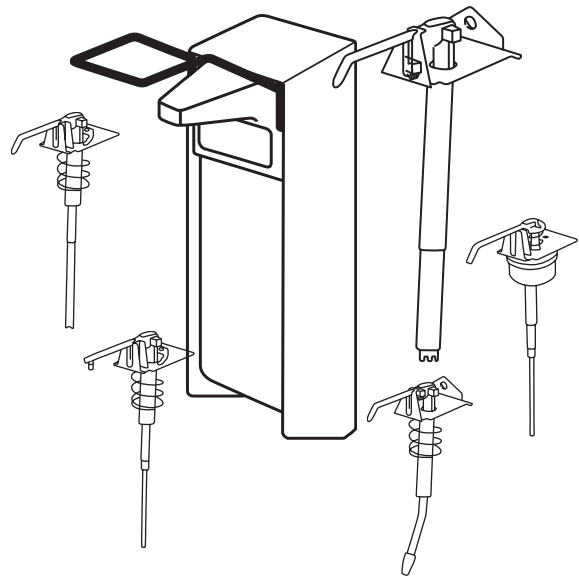
Dispenser for soap and disinfectant
 with interchangeable pump units

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


The compatibility of filling material and dispenser system must be checked prior to the use of new filling materials, in order to be able to guarantee fault-free functionality. The manufacturer assumes no liability for any damages or costs which arise due to the incompatibility of filling material and dispenser systems.

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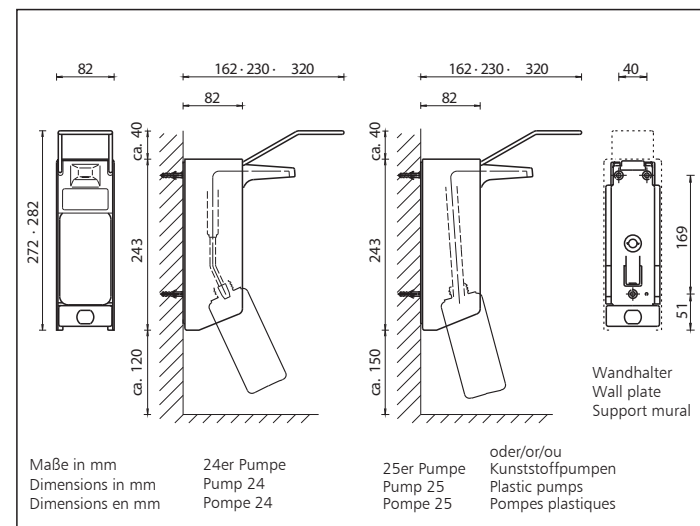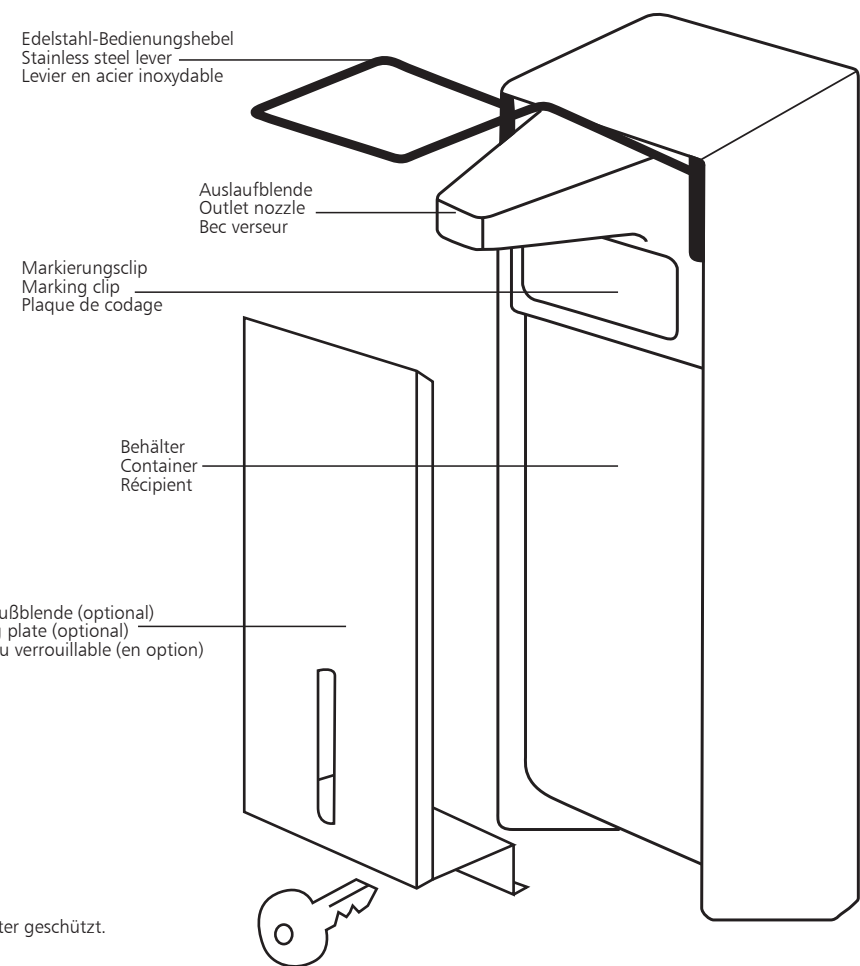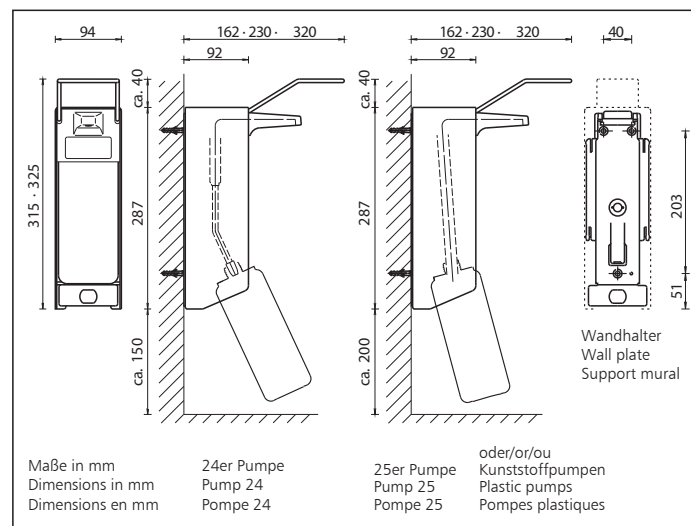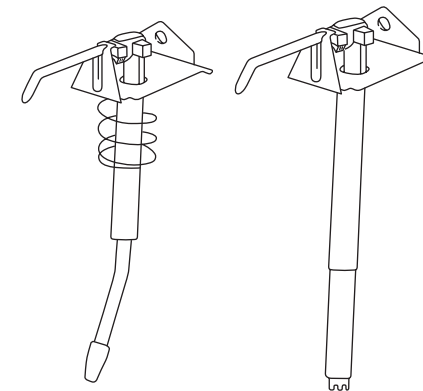
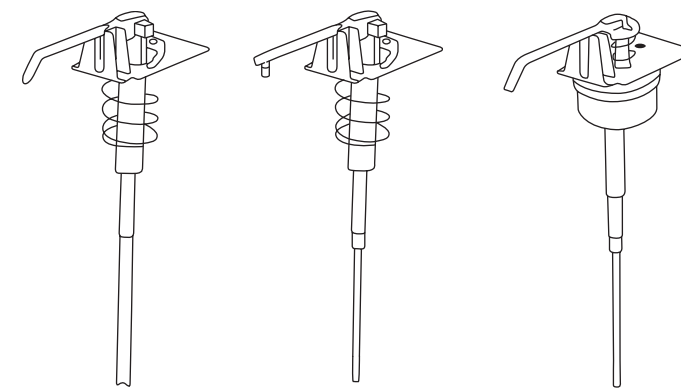
Range 24 pump with
 crooked stainless steel
 sucking tube,
 autoclavable

Range 25 pump with
 straight stainless steel
 sucking tube,
 autoclavable

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Plastic pump with
 straight plastic sucking
 tube,
 non autoclavable

Spray pump for
 disinfectant,
 plastic
 non autoclavable

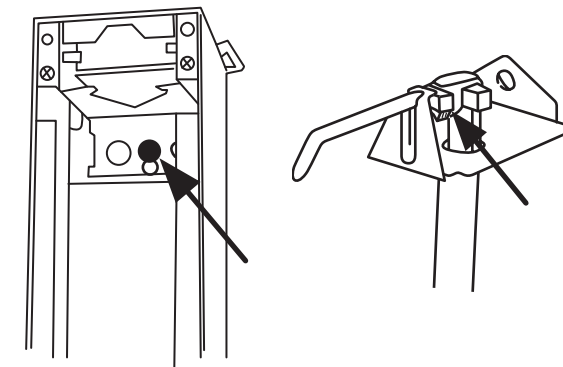
Pump for foamy soap,
 plastic,
 non autoclavable

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
Each unit is colour coded inside of the chassis. The pumps are marked by a coloured dosage wheel (except foam pump, see below). The colour indicated on the base unit must be same as of the dosage wheel.

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



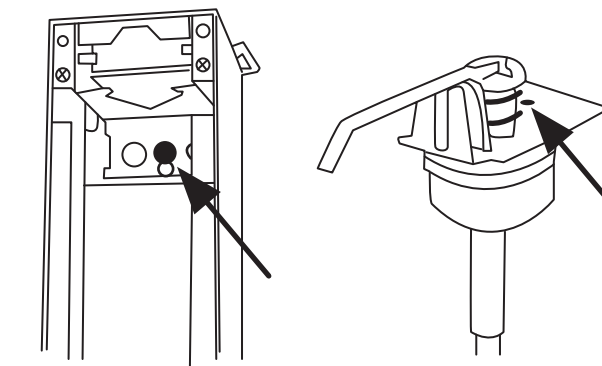
Each unit is colour coded inside of the chassis, also the foam pump. The colour indicated on the base unit must be same as on the foam pump.

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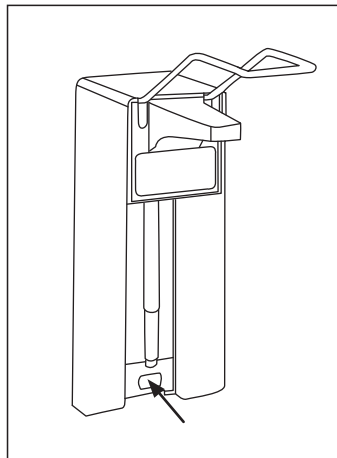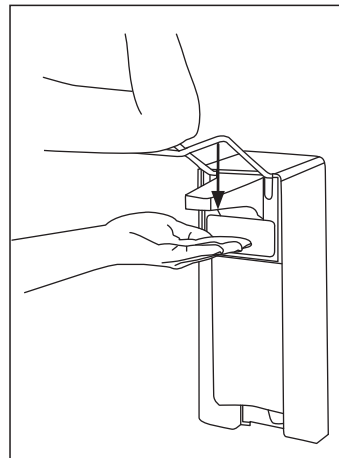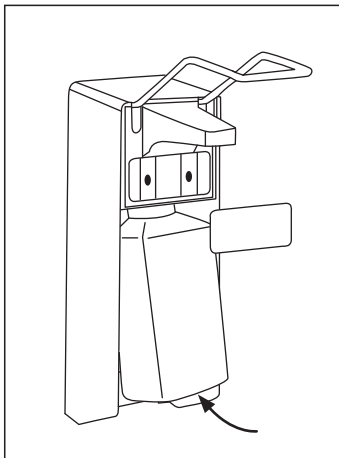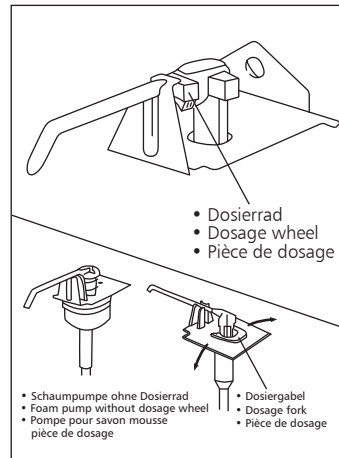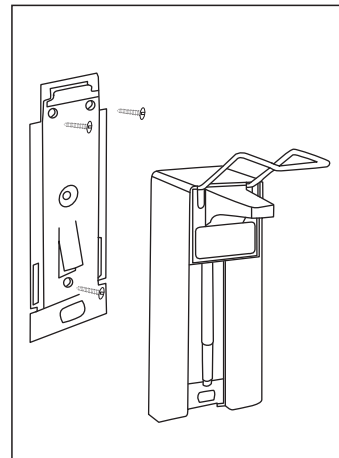Dosage

The dispenser is adjusted to the maximum amount of approx. 1.5 ml/stroke.

If required, the dosage can be reduced to a minimum of approx. 0.5 ml/stroke by gradually turning the dosage wheel head.

The plastic pump is adjusted to the maximum amount of approx. 1.5ml/stroke. If required, the dosage can be reduced to a minimum of approx. 0.8 ml/stroke by gradually turning the dosage fork head.

The foam pump has not a dosage wheel. The dosage is not changeable.

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Container exchange

Insert the refilled container from below and press the facial clip into position.

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Operation

For initial working, the unit must be operated several times until it delivers fully. Hold the open hand under the outlet nozzle.

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Cleaning/Sterilizing

Press locking device and push dispenser upwards. Dispensers using aluminium or stainless steel for units fitted with a stainless steel pump or stainless steel pumps alone (also the range 28pump with flexible sucking tube) are autoclavable up to 121 °C, 1 Bar.

Plastic pumps and powder coated dispensers are not autoclavable!

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Maintenance

The Robert-Koch Institute recommends: **“Dispensers for soap and disinfectants must be thoroughly cleaned or sterilised whenever the cartridges are changed”** [Source: see above “Quelle”]. – Though the pump is made from stainless steel, rust can occur. This can be prevented by observing the above recommendation. – Routine cleaning with warm water and soap, do not use any abrasive cleansers or such containing chlorine.

Do not dismantle the unit. Use original spare parts only. Attention! If you fail to use the right liquid, it can splash into your eyes.

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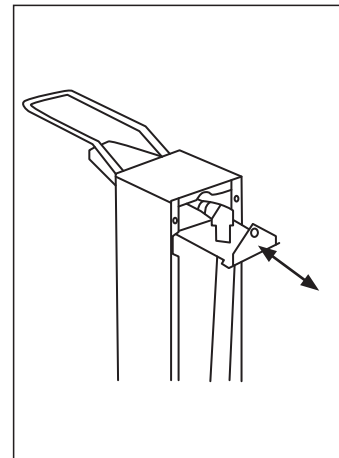
Pump exchange

Make sure that the colour code inside of the chassis is same as colour of the dosage wheel.

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
Exchange of stainless steel pump

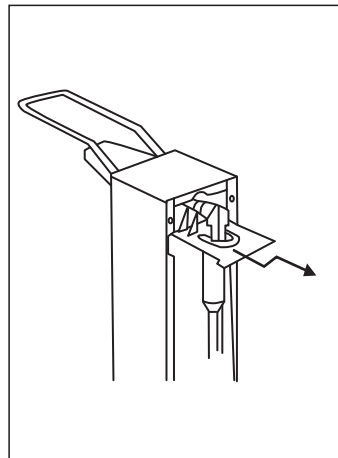Hold operating lever up and pull out stainless steel pump from backwards. For inserting the stainless steel pump hold operating lever up and place pump into position.

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Exchange of plastic pump

Hold operating lever up and slightly lift up the plastic pump and pull it out. For inserting the plastic pump hold operating lever up and place pump into position.

Check before installation

1. The dispenser should be fitted above a sink so that wet hands can drip.
2. Required space above the dispenser is 40 mm; for below refer to dimensions overleaf.
3. Required height: 130–135 cm, from floor to top of dispenser.

Mounting

Remove the container. Slightly press the fixing catch and pull down the wall plate.

Fix wall plate with enclosed plugs and screws. Mount the dispenser onto the wall plate by firmly pressing until it clicks into place.

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