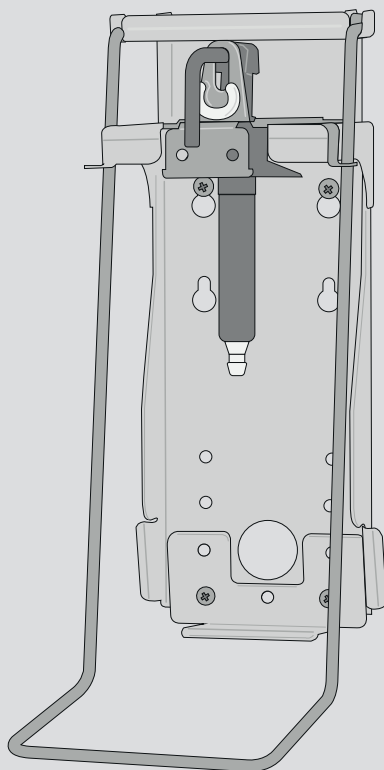
Handbuch

Manual

AIR-WOLF

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**Read these instructions for use carefully and keep them in a safe place.**

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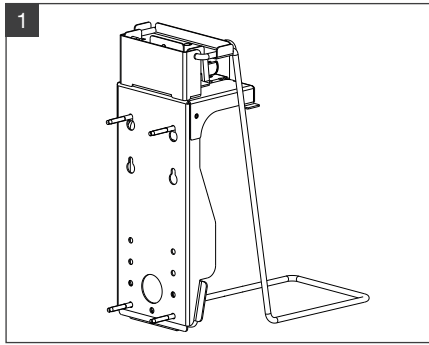
The compatibility of filling material and dispenser system must be checked prior to the use of new filling materials, in order to be able to guarantee fault-free functionality. The manufacturer assumes no liability for any damages or costs which arise due to the incompatibility of filling material and dispenser systems.

- Do not take apart the housing!
- Use only original replacement parts.
- The dispenser is fit for the dosage of suitable solutions only (such as soaps, lotions and disinfectants).
- The use of unsuitable solutions can influence the function of the dispenser and may cause the solution to spray into eyes.
- The manufacturer assumes no liability for any damages or costs which arise due to the incompatibility of filling material and dispenser systems.

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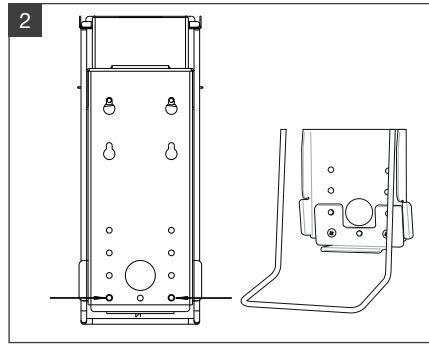
Before mounting, it must be decided whether a 0.5 liter or a 1.0 liter bottle is used.

Mount the dispenser on the wall at the top and bottom using the supplied mounting material.

The dispenser is mounted with 4 dowels and screws. Use the enclosed fastening kit for this purpose.


Required drill diameter: 6 mm

We recommend placement above or in close proximity to the sink.

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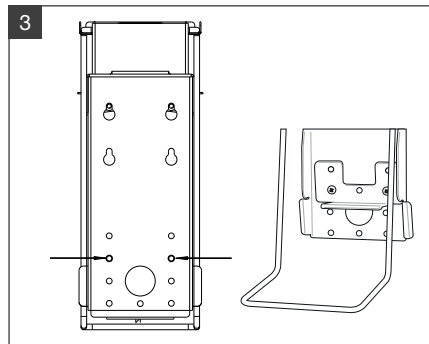Fasten the base angle for the 1.0 liter bottle in the bottom two drill holes.

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

**or**

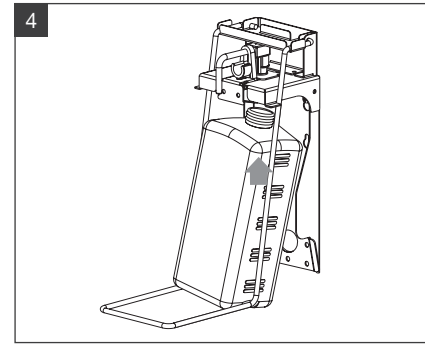Fasten the base angle for the 0.5 liter bottle in the top two drill holes

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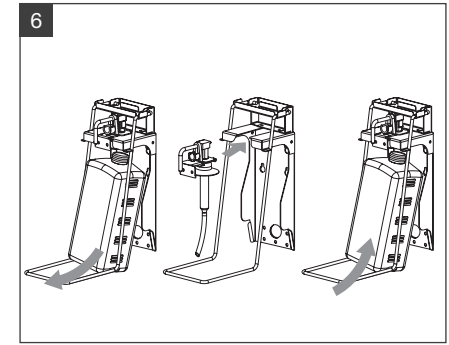
Insert a filled container into the holder from below and snap it into place at the bottom of the holder (Fig. using the 1.0 liter bottle example). Press the lever until the pump is filled.

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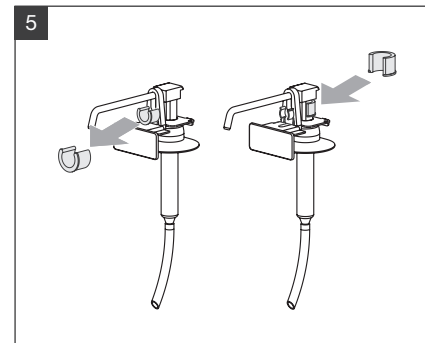To change the pump, remove the bottle, then pull the pump out to front. Push the new or cleaned pump into the guide until it clicks into place.

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
The preset dosage is approx. 1.5 ml/stroke. It can be reduced to approx. 0.9 ml/stroke by inserting the dosage device.

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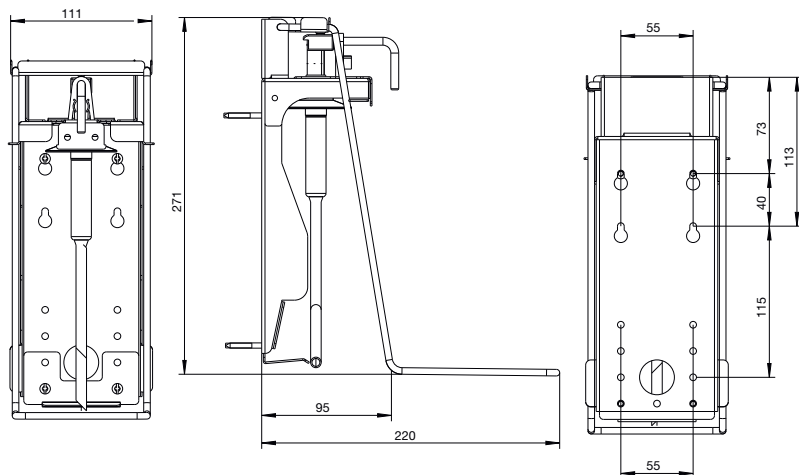### Filling quantities

The dispenser can be filled with both 0.5 liter and 1.0 liter Euro bottles. The bottle is not included in the scope of delivery.

### Adjustment on the dispenser

Base angle position individually adjustable for bottle size selection.

### Suction hose

The soap and disinfection dispenser is supplied with a 130 mm suction hose, suitable for a 1.0 liter Euro bottle.

For the use of the 0.5 liter bottle, the hose can be shortened accordingly.

### Additional products

Spare or additional products such as empty bottles or suction hoses can be found at [www.air-wolf.de](http://www.air-wolf.de):

- 1.0 liter bottle (article 37-303)
- 0.5 liter bottle (article 37-500)
- Suction hose, 130 mm for 1.0-bottle (article 37-451)
- Suction hose. 90 mm for 0.5 liter bottle (article 37-536)

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### Cleaning/Sterilization

The dispenser and stainless-steel pumps (outlet pipe made from stainless-steel), are autoclavable with superheated steam (121 °C, 2 bar, max. 20 min dwell time).

- In order to guarantee proper functioning of the dispenser regular manual cleaning or processing in cleaning-disinfection devices is necessary. Manual cleaning: Regularly remove adherent product residual. Cleaning of the dispenser housing and the pump with hot water. Rinsing the pump with hot water. Disinfection of shroud is possible with appropriate surface disinfectants.
- Processing in cleaning-disinfection devices: choose program „Dispenser cleaning“ and use pH-neutral cleaning solutions. Important: Cleaning of plastic pumps up to max. 65 °C!
- Processing in autoclave: the dispenser and stainless steel pump are autoclavable with superheated steam (121 °C, 2 bar, max. 20 min dwell time). Plastic wall plate, bottle, plastic pumps (K pumps) are not autoclavable.

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### Attention!

Dispose of the packing material into the corresponding recycling plants. If you definitely stop using the appliance please contact the next recycling center or ask your specialized dealer about the applicable disposal regulations.

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AIR-WOLF GmbH